MD

STERILE R

**Sunmatic The Safety Company AB**

Sörredsvägen 113 | 418 78 Göteborg | Sweden
+46 31 799 16 00 | info@housegard.com
www.housegard.com

Contains CE-approved
medical products.

RFX+CARE®

RFX+CARE Manufacturing Co., Ltd.
No. 599 Wuxie Road Yuecheng District Shaoxing,
Zhejiang, China 312000

EU REP

RFX+Care International A/S
Gammel Strandvej 22B, 2990 Nivå, Denmark

WOUND CLEANSING WIPES x20*18 x 12,5 cm*

Intended use: For cleaning the skin and minor wounds and abrasions. Can also be used for cleaning hands before treating an injury. They contain a mild and natural salt solution, but are not antibacterial.

Instruction for use: Clean the wound from the centre outwards to avoid getting bacteria into the wound. Throw the swab away after use.

Warning: External use only. Single use only. Never use if the package is damaged. Check the integrity of the individual sachet before use. In case of repeated discomfort, ask your doctor for advice. If swallowed, contact a doctor immediately and show the packaging. Use before the expiry date on the sachet. Store in a cool, dry place. After use, do not dispose of the compress in the environment.

Note! Instead of wipes you can use soap and water or an antibacterial solution. Soap and water are recommended.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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